

# **CITO THAW**

INSTRUCTION MANUAL

## TO USE



LIFT OUT  
BASKET



THAW MONITOR

1. Fill Cito Thaw with warm clean tap water.
3. Agitate the water by taking the basket and moving it up and down several times to assure water temperature is consistent through the water bath.
4. Place THAW MONITOR in water for a minimum of 10 seconds. See instructions on back of THAW MONITOR to determine temperature of water bath.
5. To achieve proper temperature, mix warm and cold water.
6. With the correct water temperature you are now ready to thaw semen in your Cito Thaw.

Thank you for selecting the Cito Thaw. Cito Thaw was designed to give you years of trouble free service.

## TWO YEAR WARRANTY

Any Cito Warm Water Thaw Device found to be defective in material or workmanship within 2 years from date of purchase will be repaired or replaced free of charge, when shipped prepaid to the factory. Warranty does not include breakage of the thermos liner.

**Note: No thermos liner in AC voltage models.  
Model AI-CT-120 and AI-CT-220 Do not use with a portable Generators.**

*Keep Electrical Connections Clean and Dry*

### SHIPPING INSTRUCTIONS

Package to avoid damage, label

**FRAGILE - GLASS**

and ship to:



**PRODUCTS, INC.**

N8779 COUNTY ROAD X • P.O. BOX 90  
WATERTOWN, WI 53094 U.S.A.

TEL: (920) 261-6646 • FAX: (920) 261-1350

[www.citoai.com](http://www.citoai.com) • [sales@citoai.com](mailto:sales@citoai.com)