

TWO YEAR WARRANTY

Any Cito Warm Water Thaw Device found to be defective in material or workmanship within 2 years from date of purchase will be repaired or replaced free of charge, when shipped prepaid to the factory. Warranty does not include breakage of the thermos liner.

SHIPPING INSTRUCTIONS

Package to avoid damage, label

FRAGILE - GLASS

and ship to:



PRODUCTS, INC.

N8779 COUNTY ROAD X • P.O. BOX 90

WATERTOWN, WI 53094 U.S.A.

TEL: (920) 261-6646 • FAX: (920) 261-1350

www.citoai.com • sales@citoai.com

CT-I-04-B-XX-2K



INSTRUCTION MANUAL

AI-CT-MAN-F
AI-CT-MAN-C

CITO THAW - F TEMP. READ OUT
CITO THAW - C TEMP. READ OUT



THAW MONITOR



Thank you for selecting the Cito Thaw. Cito Thaw was designed to give you years of trouble free service.

TO USE

1. Fill Cito Thaw with warm clean tap water.
3. Agitate the water by taking the basket and moving it up and down several times to assure water temperature is consistent through the water bath.
4. Place THAW MONITOR in water for a minimum of 10 seconds. See instructions on back of THAW MONITOR to determine temperature of water bath.
5. To achieve proper temperature, mix warm and cold water.
6. With the correct water temperature you are now ready to thaw semen in your Cito Thaw.

TO USE

1. Fill Cito Thaw with warm clean tap water.
3. Agitate the water by taking the basket and moving it up and down several times to assure water temperature is consistent through the water bath.
4. Place THAW MONITOR in water for a minimum of 10 seconds. See instructions on back of THAW MONITOR to determine temperature of water bath.
5. To achieve proper temperature, mix warm and cold water.
6. With the correct water temperature you are now ready to thaw semen in your Cito Thaw.

TWO YEAR WARRANTY

Any Cito Warm Water Thaw Device found to be defective in material or workmanship within 2 years from date of purchase will be repaired or replaced free of charge, when shipped prepaid to the factory. Warranty does not include breakage of the thermos liner.

SHIPPING INSTRUCTIONS

Package to avoid damage, label

FRAGILE - GLASS

and ship to:

CITO PRODUCTS, INC.

N8779 COUNTY ROAD X • P.O. BOX 90
WATERTOWN, WI 53094 U.S.A.

TEL: (920) 261-6646 • FAX: (920) 261-1350

www.citoai.com • sales@citoai.com

CT-I-04-B-XX-2K

CITO THAW INSTRUCTION MANUAL

AI-CT-MAN-F
AI-CT-MAN-C

CITO THAW - F TEMP. READ OUT
CITO THAW - C TEMP. READ OUT



THAW MONITOR



Thank you for selecting the Cito Thaw. Cito Thaw was designed to give you years of trouble free service.